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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Nimbo walker is a walking aid for people with reduced mobility. It is designed for indoor use and folds away for easy storage and transportation. It can be used outdoors on flat surfaces.

SAFETY PRECAUTIONS


Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 45 kg (size 1), 65 kg (size 2), 85 kg (size 3) and 110 kg (size 4).
- Make sure that all parts (wheels, cables and handles) are secure and in good working order.
- Do not stand or sit on the edge of the seat as this could damage the product.
- Do not use the walker on stairs or escalators.
- Only use the accessories and parts supplied by the manufacturer.
- The walker is a walking aid. It is not a wheelchair. It was not designed to carry a person sitting on it. Such use could damage the wheels and the structure of the walker itself. As a result, the walker could become brittle and no longer meet its mechanical specifications when used in the future. Only use the walker as a walking aid.
- Do not use the walker to walk backwards.
- Do not make any adjustments to the walker while it is in use.
- Check that the brakes are working properly before use.
- All wheels must always be in contact with the ground.
- This walker is equipped with a directional locking system and an anti-roll-back mechanism to ensure safe and stable use.
- Make sure you put your hands on each handle so that the walker can be in the parallel force position.
- Take care when climbing kerbs or when using inclined, uneven or slippery surfaces. Failure to do so may result in a serious risk of falling or injury.
- Do not use the walker on wet, icy or snowy slopes.
- If any parts are damaged, broken or worn, contact your dealer.

MOUNTING & USE


Instructions for use:

1. Remove the Nimbo from its packaging.
2. To unfold the Nimbo, pull the handles and the handbrake in opposite directions. Continue unfolding until the pins click into place.
3. To install the legs, match the leg numbers with the corresponding frame numbers. Depress the leg push pins and slide them into the frame. Once in position, the push pins will engage through the frame.
4. Adjust the height of the walker by pressing the pins on each leg and sliding the leg to the desired position.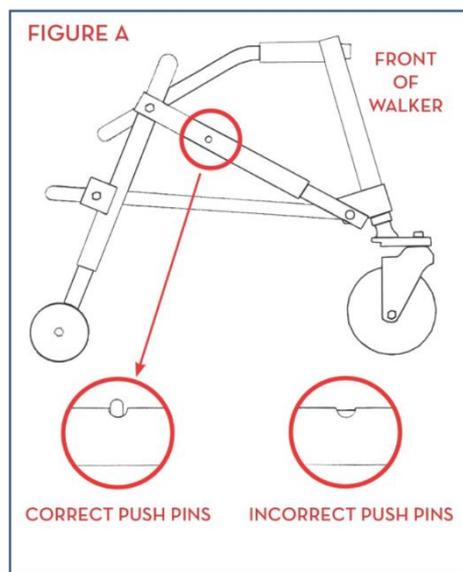


Information for the user:

1. Lift and position knob (A) to put the front wheels in the swivel or locked position.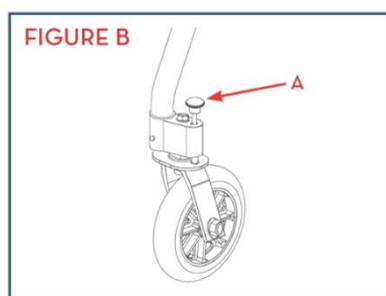

2. To unfold the Nimbo, face the open end, press the push pins, tilt the walker so that it rests only on its rear feet and push the front feet towards the rear feet.
3. To cancel the one-way function of the rear wheels, lift the aluminium pin and insert it into the cancellation bracket.

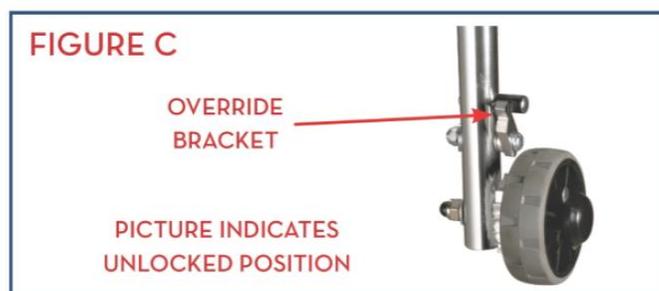CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the walker regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth.
- Make sure there is no build-up of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check all screws and bolts for wear and strength. Replace all broken, damaged or worn parts immediately.
- In the event of a malfunction, do not repair the rollator yourself. Consult an authorised dealer.

Checklist :

To keep your walker in good condition, check before each use and regularly as follows.

Check the item	Anytime	Weekly	Monthly	Yearly
Clip for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable, and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Model	Size 1	Size 2	Size 3	Size 4
Handle height	41-51cm	51-68 cm	59-80,5 cm	75-94,5 cm
Width between handles	34,5 cm	36,5 cm	38,5 cm	41,5 cm
Footprint	50 x 56 cm	58 x 61,5 cm	59 x 73 cm	66,5 x 77,5 cm
Weight	3,8 kg	4,3 kg	5,4 kg	6 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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